



Check for signs of Sudden Cardiac Arrest: ☒ Unresponsive
☒ Not Breathing Normally


1 PULL

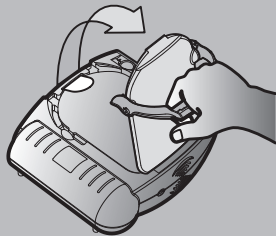

2 PLACE

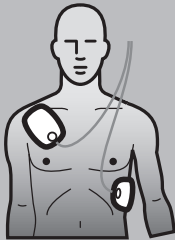


3 PRESS

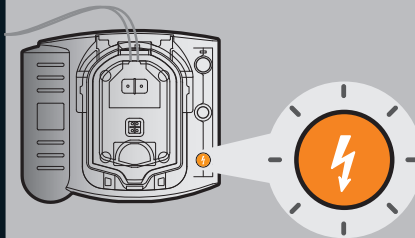


QUICK REFERENCE GUIDE

DETAILED INSTRUCTIONS



To start: Pull handle. Remove protective cover.

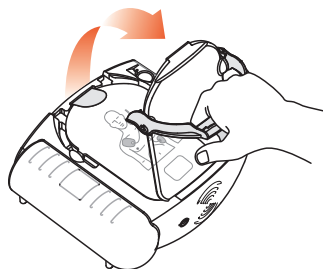



Remove all clothes from victim's chest.
Clean and dry skin, if needed.

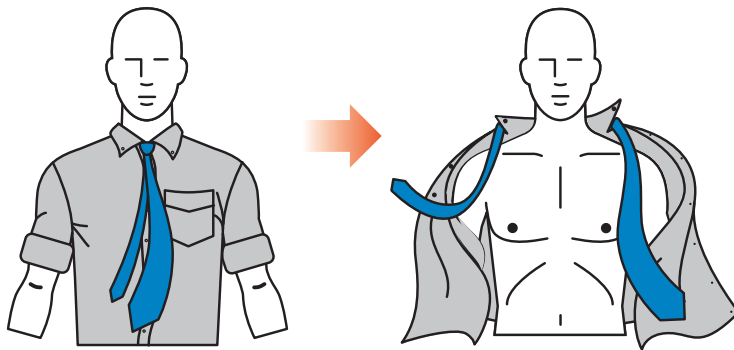



Peel off film seal and remove pads.

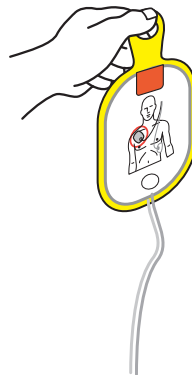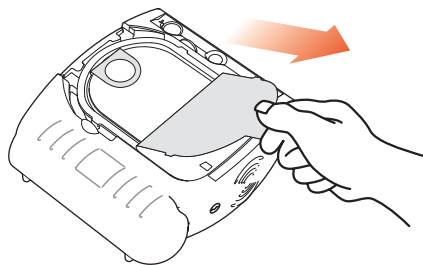

Peel each pad from yellow plastic liner.

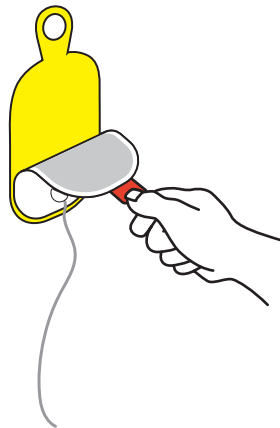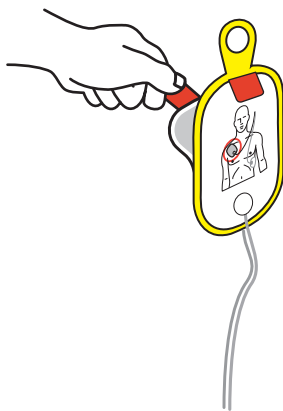


Place pads exactly as shown in the picture.

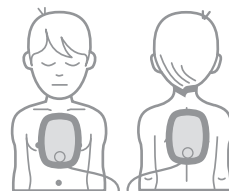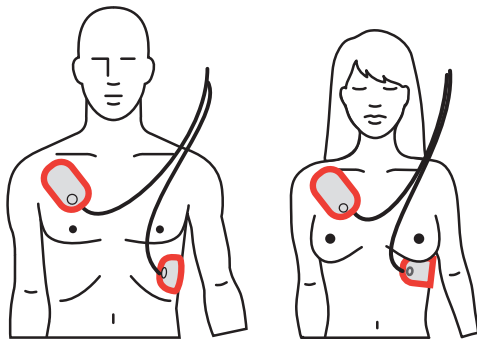


(0-8 YEARS) < 55 lbs/25 kg


Analyzing ... Do not touch the patient!

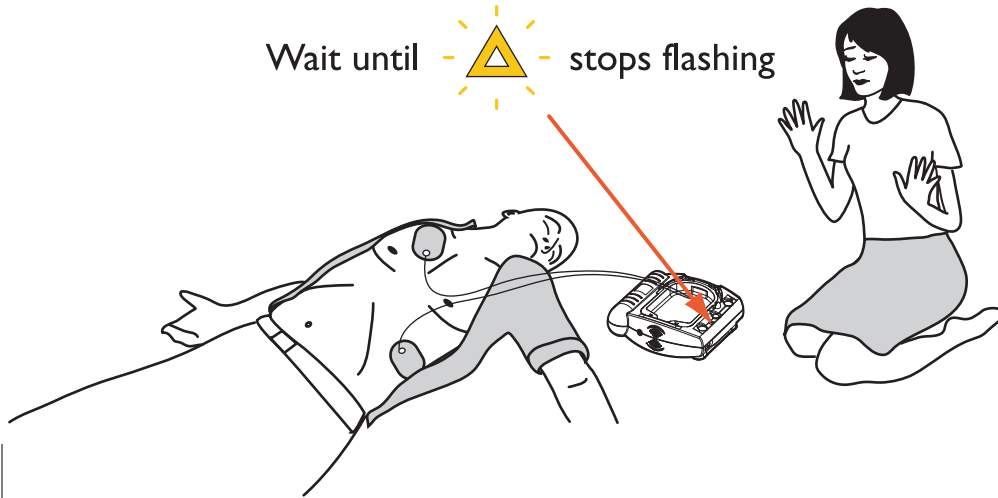

To shock: Press flashing **ORANGE** shock button.
Do not touch patient.

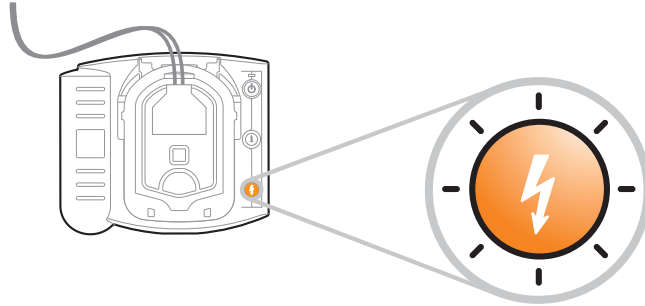


To get help with CPR: Press flashing **BLUE** button.

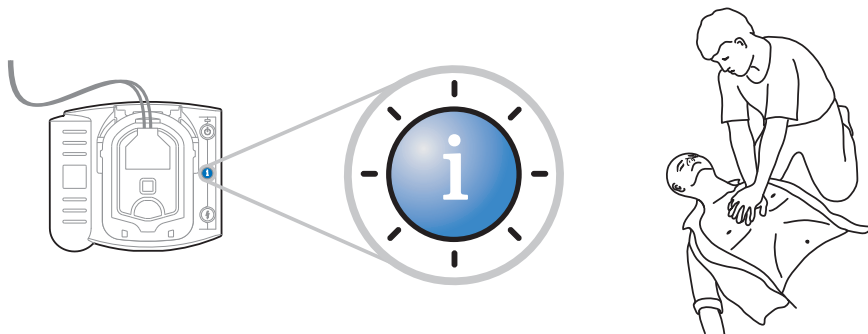


30 compressions • Pinch nose, tilt head • 2 full breaths

To turn ON: Press **GREEN** on/off button.

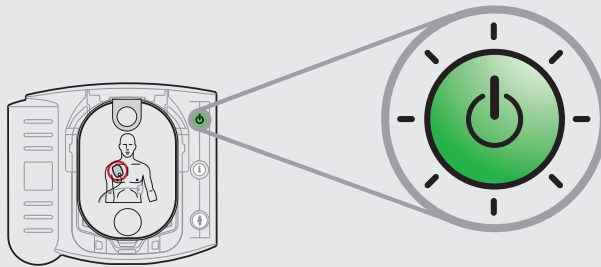

If cartridge needs to be replaced: Slide latch.

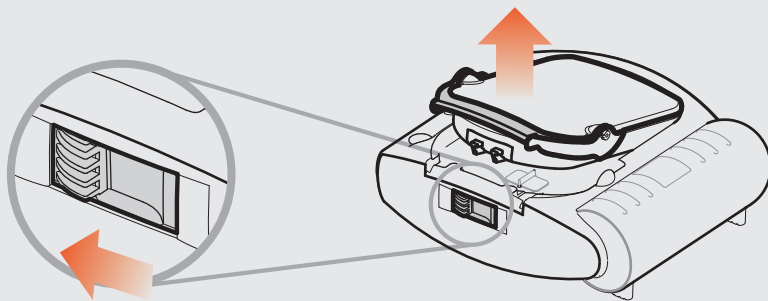

Printed in USA

Philips Medical Systems
Bothell, WA 98021-8431, USA

Copyright © 2010
Philips Electronics North American Corp

453564141542 REV III

REF M5066-97800



453564141542

PHILIPS