
PROFLEX® WRIST SUPPORTS


400/405

- 1. Wrap strap around entire wrist.
- 2. Secure in place with the hook-and-loop closure. Use on right or left hand.

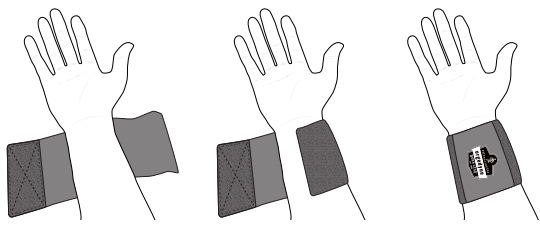
415

- 1. Feed strap through grommet.
- 2. Slip over hand to wrist just below palm.
- 3. Wrap strap around wrist, until the level of support feels comfortable. Use on right or left hand.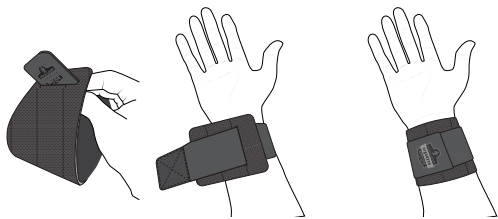


425

- 1. Insert thumb into hole.
- 2. Wrap support past palm, over the top of hand.
- 3. Loop around the wrist to secure.
- 4. Alternatively, wrap across the top of hand and then past palm before looping around the wrist to secure. Use on right or left hand.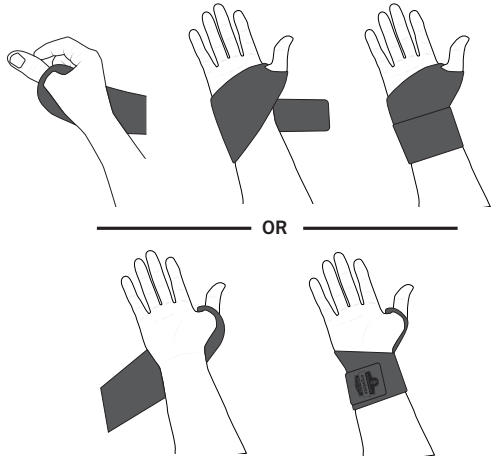


660

- 1. Insert hand into single opening end of support, feeding thumb through the small hole and remaining fingers through the larger opening.
- 2. Tug at base of wrist until material is even. Use on right or left hand.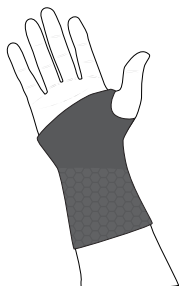


670/675

- 1. Slip hand and thumb through opening.
- 2. Place removable strap with loop to hook so the strap lays away from the body.
- 3. Pull strap underneath to wrist side and then over the back of the wrist.
- 4. Fasten hook-and-loop closure(s) to secure in place. Use on right or left hand.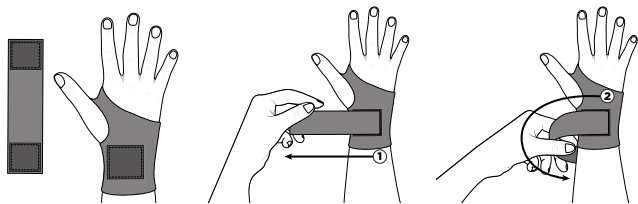


680/685

- 1. With the elastic loops undone, insert hand into single opening end of support, feeding thumb through the small hole and remaining fingers through the larger opening.
- 2. Tug at base of wrist until material is even.
- 3. Wrap elastic strap(s) around wrist until the level of support feels comfortable. Use on right or left hand.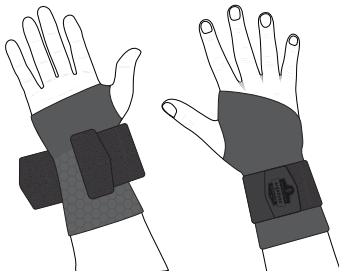


4000/4010

- 1. Place around wrist (right or left, as stated) aligning the wrist to the orange line indicator.
- 2. Fasten hook-and-loop closure on back of wrist.
- 3. Place palm strap between thumb and index finger. Attach end of strap to hook-and-loop closure on back of wrist.
- 4. Wrap elastic strap across top of the wrist and fasten at desired tension. Right or left hand support must be used on correct hand.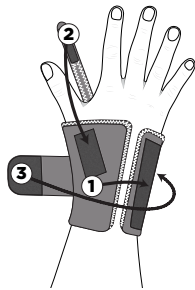


4005/4015

- 1. With the elastic straps undone, insert hand into single opening end of support, feeding your thumb through the small hole and remaining fingers through the larger opening.
- 2. Tug at base of wrist until thumb is completely through.
- 3. Wrap elastic strap(s) around wrist until the level of support feels comfortable. Right or left hand support must be used on correct hand.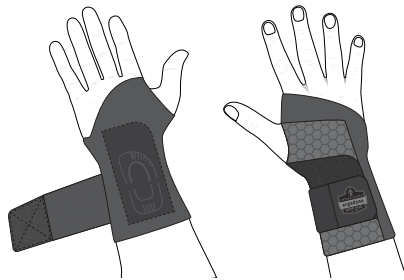

4020

- 1. Loosen closure.
- 2. Slide hand (left or right, as stated) through bottom opening of wrist support. Position palm strap between thumb and index finger.
- 3. Pull closure tab up and toward the inside of the wrist to meet hook-and-loop closure. Fasten at desired tension. Right or left hand support must be used on correct hand.

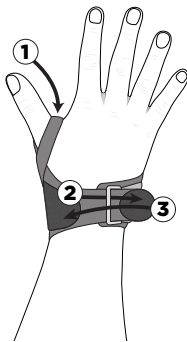CARE INSTRUCTIONS:

Machine wash cold; air dry.

* We cannot guarantee wearing personal protective equipment will prevent injury.

MANUFACTURER

Ergodyne
1021 Bandana Boulevard East, Suite 220
Saint Paul, MN, USA 55108
+1.800.225.8238

AUTHORIZED UK REPRESENTATIVE

Authorised Rep Compliance Ltd.
Arc House, Thurnham
Lancaster, LA2 0DT, UK
Tel. +44 1524 805288

AUTHORIZED EU REPRESENTATIVE

Authorised Rep Compliance Ltd.
Ground Floor, 71 Lower Baggot Street
Dublin, D02 P593, Ireland
Tel. +353 151 33758

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